

EXHIBIT I-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
INTOXILYZER MODEL 9000

PERIODIC MAINTENANCE AND STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Michael G. T. Bawit AGENCY Goodyear Police
DATE 12/04/2004 TIME 1103
INTOXILYZER SERIAL # 90-003795

☒ 1. Ensure that gas tank is attached and contains a standard alcohol concentration 0.100 AC

DIAGNOSTIC TESTS

- ☒ 1. Clock time check
☒ 2. Date check

OPERATIONAL TESTS

- ☒ 1. Deficient Subject Test (Proper Sample Recognition)
Deficient Sample printed
- ☒ 2. Alcohol-free Subject Test (Proper Sample Recognition):
0. 000 AC
- ☒ 3. Mouth Alcohol Subject Test (Proper Sample Recognition):
Invalid Sample – Begin new deprivation period printed
- ☒ 4. Radio Frequency Interference Test (Error Recognition):
RFI Detect printed
- ☒ 5. Standard Calibration Check:
0. 098 AC
- ☒ 6. Air Blanks Completed
- ☒ 7. Timer Reset

Not a Successfully Completed Test Sequence will be printed

Instrument is operating properly and accurately. YES X NO

COMMENTS:

SIGNATURE [Signature] 1144

ARIZONA FORENSIC BREATH ALCOHOL
ANALYTICAL REPORT

11 N 145TH AVENUE

INSTRUMENT INFORMATION

Analytical Instrument: INTOXILYZER 9000
Serial Number: 90-003795
Software Version: 9439.01.00
Analytical Report Number: 37951205240716
QAS: MIYAZATO, KORY
QAS Permit #: 30215
Agency: GOODYEAR PD
Last 31-Day Check: 11/12/2024
Last Annual Maintenance: 10/02/2024

SUBJECT INFORMATION

Test Date: 12/05/2024
Name: 31-Day Check,
Driver's License #:
Weight:
Gender:
State of Issue:
Date of Birth:
Driver's License Expiration:
Age:
Deprivation Start Time: 10:45
15-Minute Deprivation: Yes

OPERATOR INFORMATION

Name: GUILBAULT, MICHAEL
Agency: GOODYEAR PD
Permit #: 30305

STANDARD INFORMATION

Standard Value: 0.100
Standard Lot #: 302-402972207
Expiration Date: 02/20/2026
Bottle #: 19
Last Changed By:
MIYAZATO, KORY
Permit #: 30215

Test	g/210L	Time
Air Blank	0.000	11:03:33
Diagnostic Test	PASS	11:04:09
Air Blank	0.000	11:04:45
Calibration Chk	0.099	11:05:07
Air Blank	0.000	11:05:47
Subject Test 1	DEF*	11:09:09
Air Blank	0.000	11:09:56
Air Blank	0.000	11:10:36
Subject Test 2	0.000	11:11:01
Air Blank	0.000	11:11:43
Wait		11:15:47
Air Blank	0.000	11:16:24
Subject Test 3	INV**	11:16:39
Air Blank	RFI***	11:17:00

*Deficient Sample

**Invalid Sample - Begin new deprivation period

***RFI Detect

Intoxilyzer - Alcohol Analyzer
Model 9000

SN: 90-003795

Test Date: 12/05/2024

Test Time: 11:17

Location:

11 N 145TH AVENUE

Software Version: 9439.01.00

Standard Lot #: 302-402972207

Standard Value: 0.100

Expiration Date: 02/20/2026

Bottle #: 19

Last Changed By: MIYAZATO, KORY

Permit #: 30215

Operator: GUILBAULT, MICHAEL

Permit #: 30305

Agency: GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:17:55
Calibration Chk	0.098	11:18:17
Air Blank	0.000	11:18:57

No RFI Detected

Not a Successfully Completed Test Sequence

State of Issue:
Date of Birth:
Driver's License Expiration:
Age:
Deprivation Start Time: 10:45
15-Minute Deprivation: Yes

OPERATOR INFORMATION

Name: GUILBAULT, MICHAEL
Agency: GOODYEAR PD
Permit #: 30305

STANDARD INFORMATION

Standard Value: 0.100
Standard Lot #: 302-402972207
Expiration Date: 02/20/2026
Bottle #: 19
Last Changed By:
MIYAZATO, KORY
Permit #: 30215

Test	g/210L	Time
Air Blank	0.000	11:03:33
Diagnostic Test	PASS	11:04:09
Air Blank	0.000	11:04:45
Calibration Chk	0.099	11:05:07
Air Blank	0.000	11:05:47
Subject Test 1	DEF*	11:09:09
Air Blank	0.000	11:09:56
Air Blank	0.000	11:10:36
Subject Test 2	0.000	11:11:01
Air Blank	0.000	11:11:43
Wait		11:15:47
Air Blank	0.000	11:16:24
Subject Test 3	INV**	11:16:39
Air Blank	RFI***	11:17:00

*Deficient Sample

**Invalid Sample - Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

RESULTS

EXCEPTION MESSAGES

* Deficient Sample

** Invalid Sample - Begin new deprivation period

*** RFI Detect

All air blank results must be 0.000.

Consecutive subject test results must not differ
by more than 0.020 g/210L.

Standard check results must be $\pm 10\%$.